# Instruction Manual **2**

## Automatic Blood Pressure Monitor

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### Model HEM-7143T1

All for Healthcare

#### Read Instruction manual ① and ② before use.

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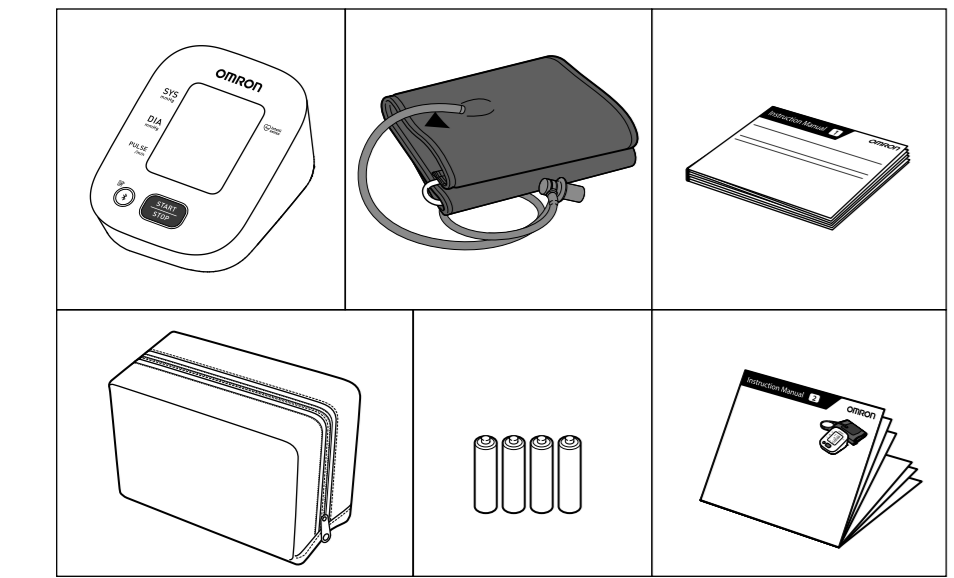
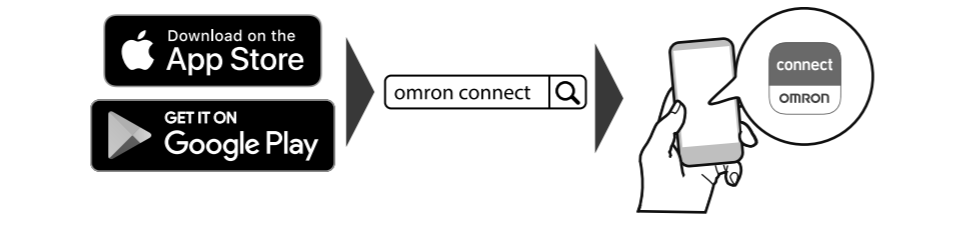
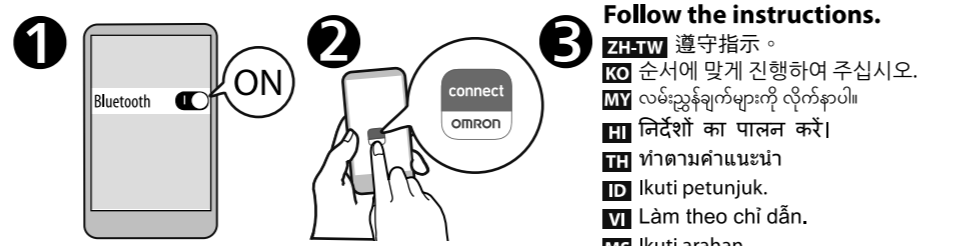
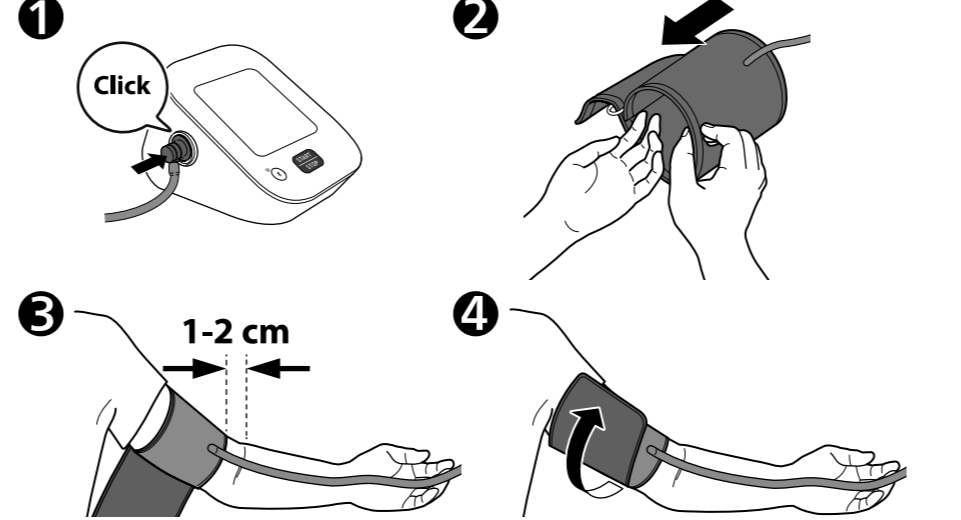
#### 3 Tube side of the cuff should be 1 - 2 cm above the inside elbow.

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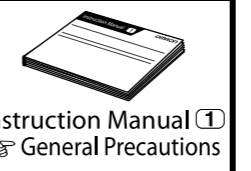
#### 4 Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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
#### If taking measurements on the right arm, refer to:

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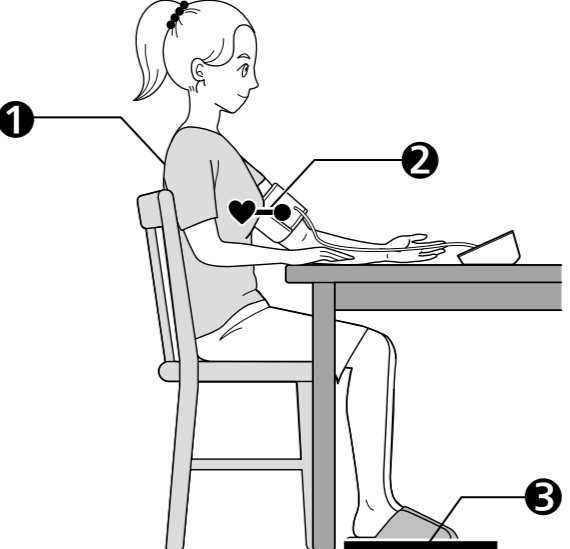#### 1 Sit comfortably with your back and arm supported.

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#### 2 Place the arm cuff at the same level as your heart.

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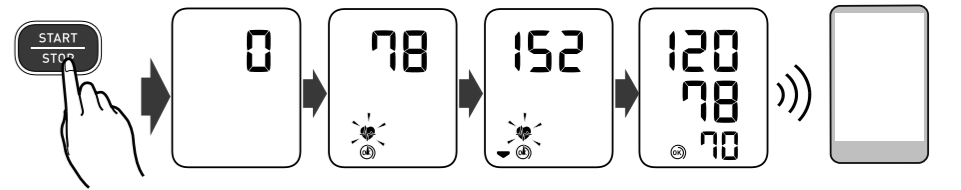
#### 3 Keep feet flat, legs uncrossed, remain still and do not talk.

- ZH-TW** 雙腳平放，不要交叉雙腿，保持靜止，不要說話。
- KO** 다리를 꼬지 않은 상태에서 양발은 바닥에 대고, 움직이거나 말을 하지 않습니다.
- MY** မြဲတင်းချထားကာ မြဲထောက်ချိတ်ထားခြင်းမရှိဘဲထိုင်ကာ မြဲငြိမ်နေပြီး စကားမပြောပါနှင့်
- HI** पैर सपाट रखें, पैर मुड़े न हों, स्थिर रहें और बात नहीं करें।
- TH** วางเท้าราบกับพื้น ไม่ไขว่ขา นั่งนิ่งๆ และห้ามพูดคุย
- ID** Jaga agar telapak kaki rata, jangan silangkan kaki, tetap diam, dan jangan berbicara.
- VI** Giữ bàn chân phẳng, chân không bắt chéo, không cử động và không nói chuyện.
- MS** Pastikan kaki anda rata, kaki tidak bersilang, jangan bergerak dan jangan bercakap.

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#### When the [START/STOP] button is pressed, the measurement is taken and saved automatically. Open the app to transfer the reading.

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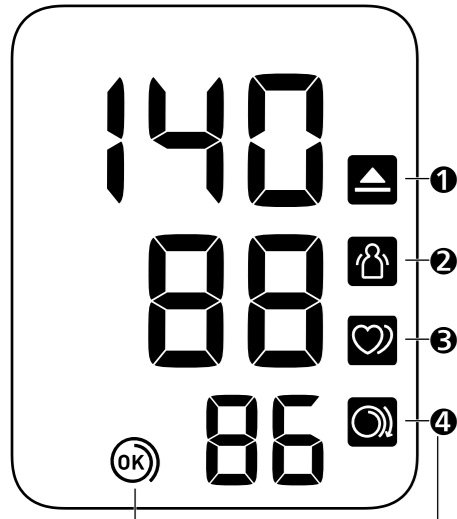
#### If your systolic pressure is more than 210 mmHg:

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### 2 Appears when your body moves during a measurement. Remove the arm cuff, wait 2 - 3 minutes and try again.

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### 3 Appears when an irregular rhythm\*\* is detected during a measurement. If it appears repeatedly, OMRON recommends to consult your physician.

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### \*The high blood pressure definition is based on the 2018 ESH/ESC Guidelines.

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### \*\*An irregular heart beat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected during a measurement.

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### Error messages or other problems? Refer to:

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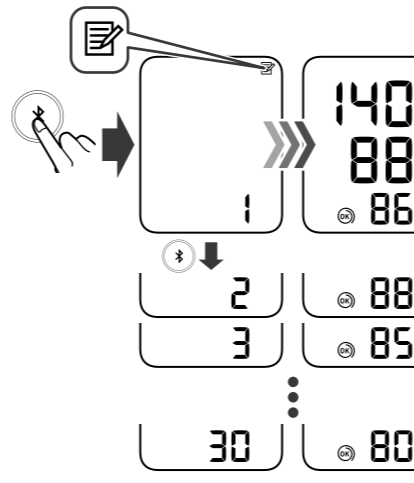
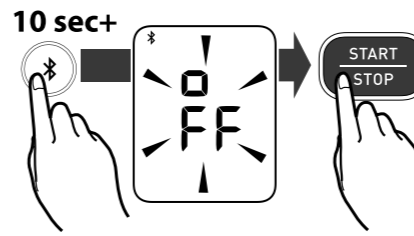
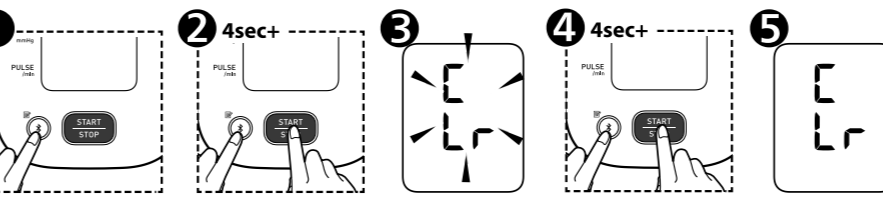
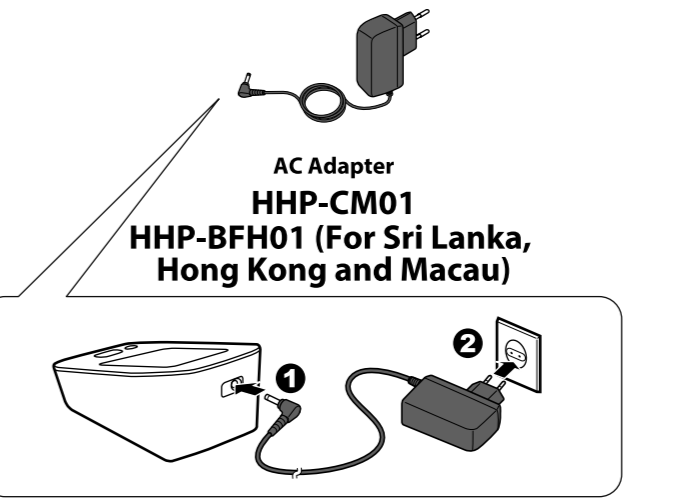
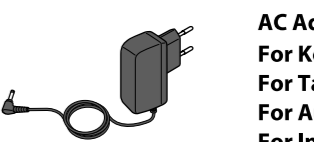
- AC Adapter**
- For Korea:** HHP-KSM01
- For Taiwan:** HHP-AM01
- For Australia and New Zealand:** HHP-OH01
- For India:** HHP-BH01

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